




SuperCanister Instructions v3.0

Novice Models with Magnetic Safety Coupling - All sizes.

Introduction

The Supercanister Novice model is a foreskin restoration tugging device. The device allows tensioning of the penis skin tube to reverse circumcision by skin expansion. The device allows the glans (head) to partially recess inside, making it easier for men with little slack skin to apply the device. Once attached, the device permits urination without removal. Each Supercanister is precision machined by ADI from a solid billet of food-grade Dupont polymer. The device incorporates an integral magnetic safety coupling which will detach in the event of excessive tension, preventing injury to the penis. Under normal levels of tension, the coupling will stay attached. The safety coupling's swiveling design will also prevent the penis from being subjected to torsion or twisting. The Supercanister features a raised lip machined on the end of the barrel. This lip helps to retain the inner wrap of tape and keep it from sliding off the device.

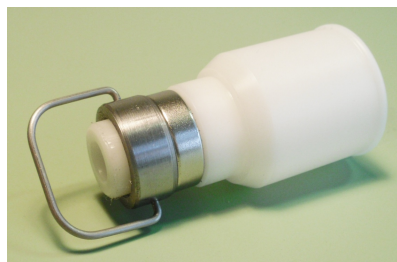

To use the Supercanister:

1. Obtain an 8" (20cm) long by 1" (2.5cm) wide strip of surgical tape.
2. Apply the tape to the end of the Super Canister, by initially sticking the tape to the barrel, then wrapping the tape back on itself so the adhesive side is out. (See Figures 1 & 2)
(Note: Instead of using tape at this step, you can also use Adhesi-Med (TM) MRS skin safe adhesive. Brush a small amount on barrel and spread evenly.)



Figure 1



Figure 2

3. Have a 12" long (30cm) by 2" (5cm) wide piece of tape on hand. Peel some of backer paper to make the tape ready to apply.

4. Place tip of glans inside canister, and roll skin over the exposed adhesive tape (or adhesive). Hold skin with a firm grip for 10 seconds until adhesive sets. (See Figure 3 & 4.)

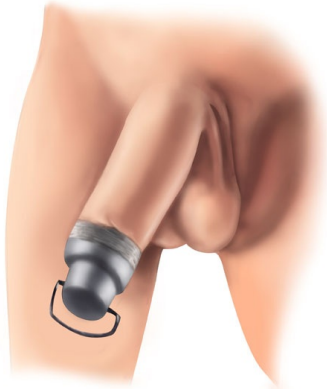


Figure 3



Figure 4

5. Begin taping the foreskin to the device. The tape should contact approximately 3/4"-1" (19-25mm) of skin, with the rest contacting the Supercanister. See Figure 5 & 6. The tape should not come in contact with the swiveling collar.

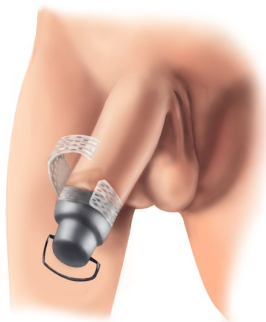


Figure 5



Figure 6

6. Continue to wrap all of the tape around the device (Figure 7 & 8.) Be sure to cover over any skin folds. Before completely wrapping, fold back the end of the tape to facilitate later removal. Wrap carefully with moderate pressure to avoid leaks.

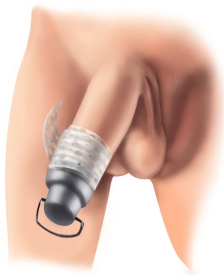


Figure 7



Figure 8

7. Apply an elastic suspender or Advanced Devices Torpedo weights to the bail, and tug with light to moderate pressure.. Our Torpedo weights and Y-Back suspenders are very effective, available on our website www.foreskin-restoration.info

8. To urinate, simply disconnect strap or move out of way, hold barrel and urinate normally. If stream is erratic, gently pull on device to align urethral opening with drain hole. Shake out remaining urine when done. Insert disposable cotton roll to absorb any remaining urine and prevent soiling of clothing.

8. If you feel discomfort, disconnect elastic or weight and take a 15 minute break.

9. To remove device, spray Citri-Med (TM) Citrus Adhesive and Tape remover onto tape. This will cause the tape to immediately release. Slowly remove tape without harming skin.

Foreskin Restoration suggestions:

Foreskin restoration can be accomplished with comfort and convenience. While you can experiment to see which is best for you, we suggest the following:

1. Tug for 8-12 hours per day, 3 to 5 days per week. Take at least 2 days per week off to allow the skin to repair and regrow.
2. Tug with moderate weight. You are using too much tension if you get bruises or skin tears. In this event, take several days off from tugging to allow the skin to heal. Re-start tugging after healing with lower tension levels.
3. Use quality tape. The tape we make available from our webstore has been tested to ensure its performance with our devices. Other types or brands of tape may not securely hold the device in place, or allow urine leakage.

Warnings:

1. Do not sleep while wearing any foreskin restoration device.
2. Do not exceed 13 ounces of tugging force.
3. If you feel any pain discontinue use immediately .
4. Consult with a medical professional before attempting foreskin restoration.
5. Wash the Supercanister inside and out with soap and water after each use.
6. Do not wear the Supercanister when operating machinery or other potentially hazardous conditions.
7. This device uses an extremely strong magnet. Do not use if you wear a pacemaker. Do not place the magnet near electronic devices or media.



www.foreskin-restoration.info

For online support join our forum at
www.foreskin-restoration.info/forum

“Helping you reach your goals”